



## Pairing Quick Guide

### Pairing Accessories – Before you Start

- Make sure you program the hearing aids first before you pair the accessories.
- Close out of the software and open and close the battery doors on hearing aids before pairing.
- Make sure you have the TV Link plugged in before pairing.
- Make sure you charge the myPAL, Remote Control 2 and Phone Link 2 accessories for at least 3 hours before pairing.

### Pairing with Remote Control 2

- Turn on Remote Control by sliding On/Off switch to the down position.
- Remove silver cover from the bottom of the RC2.
- Push the white pairing button.



- Open and Close battery doors on hearing aids within 20 seconds of pushing the pairing button.

## TV Link 2 Pairing

- Make sure the TV Link is plugged in.
- Push the pairing button on the back of the TV Link 2.



- Open and Close battery doors on hearing aids within 20 seconds of pushing the pairing button.

## Pairing Phone Link 2 with Hearing Aids

- Make sure Phone Link 2 is turned on. Hearing Instruments must be turned off (open battery doors).
- Remove the silver cover from the Phone Link 2.
- Shortly press the **white** pairing button. This activates the pairing process between the hearing aid and the Phone Link 2 for 20 seconds.



- Close the battery door of your hearing aids. Successful pairing will play a melody in the hearing aids.

### Pairing with Bluetooth Cell Phone

- Turn your Phone Link 2 ON and remove silver cap.
- Place mobile phone next to Phone Link 2 with phone on and “Bluetooth” on.
- Then press the small **Blue** Bluetooth pairing button one time on the back of the Phone Link 2 which will enable pairing mode for two minutes.



- Pairing with a Bluetooth mobile phone will depend on what cell phone is used.
- Online Bluetooth pairing guide is available.
- Access through link on website: [www.jabra.com/gnhearing](http://www.jabra.com/gnhearing)
- On the mobile phone choose to search for Bluetooth devices.
- Your mobile phone should present a list of Bluetooth devices discovered. Select “Hearing Aid Phone” from the list.
- If Pass code is required for Phone Link 2 enter 0000

## Pairing myPAL with Hearing Aids

- Remove the silver cover from the myPAL.
- Shortly press the pairing button. This activates the pairing process between the hearing aid and the myPal for 20 seconds.



- Fully open and close the battery door of your hearing aid. Successful pairing will play a melody in the hearing aids.